## ■ EXPLANATION OF SYMBOLS USED

	EN
	Professional use only
	Follow instructions for use
	Unique Device Identifier
	Medical Device
	Importer
	Temperature limit
	Keep away from sunlight
	Use-by date (expiration)
	Batch code
	Catalogue / stock number
	Package contents
	Do not use if package is damaged
	Manufacturer
	Single use only
	Translation
	Unit Dose Capsule
	Unit Dose Dispensing Gun
	Dispensing Tip
	0.85 mm Syringe / Paste

## Instructions for Use

**Dryz®**  
Gingival Hemostatic Retraction Paste

**parkell®**


300 Executive Dr., Edgewood, NY 11717 USA  
(800) 243-7446 • www.parkell.com

DA: BRUGSANVISNING  
DE: GE BRAUCHSANWEISUNG  
NL: GEbruksANWIJNING  
EL: ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ  
ES: MODO DE USO  
FI: KÄYTTÖOHJE  
FR: MODE D'EMPLOI

IT: ISTRUZIONI PER L'USO  
NL: GEbruksANWIJNING  
NO: BRUKSANVISNING  
PL: INSTRUKCJA UŻYCIA  
PT: INSTRUÇÕES DE UTILIZAÇÃO  
SV: BRUKSANVISNING

mt-g medical translation GmbH & Co KG  
Stuttgarter Straße 155 • D-89075 Ulm

UK: UKRP: Topdental (Products) Ltd,  
12 Ryefield Way, Silsden,  
West Yorkshire BD20, 0EE England UK

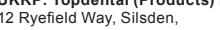Rx Only—product intended for use by a licensed dental professional

Made in USA  
A00661revK0821

## EN

### ► STORE THIS PRODUCT IN A COOL, DRY PLACE.

Refrigeration will prolong shelf life, but avoid freezing. Bring to room temperature before use.

### ► PRODUCT DESCRIPTION

Parkell's Dryz® and Dryz® Blu Gingival Hemostatic Retraction Pastes are dental products capable of temporarily controlling gingival bleeding, displacing marginal gingiva and drying the gingival sulcus around a tooth. The pastes allow the dental clinician to perform operative procedures without interference from blood, moisture or hyperplastic soft tissue.

Dryz and Dryz Blu are supplied in several ways:

1. In 0.85 gram syringes with hand plungers, or
2. In 0.24 gram single-patient use, unit dose capsules, designed for use with a Centrix®-compatible dispensing gun.

To simplify these instructions, both Original Green "Dryz" and "Dryz Blu" are referred to here as "Dryz".

### ► INDICATIONS FOR USE

Dryz Gingival Hemostatic Retraction Paste is indicated for temporary retraction and hemostasis of the marginal gingival tissues during clinical procedures such as taking subgingival and per gingival dental impressions, sealing of fixed prostheses, and restorative dentistry.

### ► CONTRAINDICATIONS FOR USE

Dryz Gingival Hemostatic Retraction Paste should not be swallowed, and is contraindicated for use on or by persons who are allergic to or sensitive to aluminum chloride, silicates or similar products. Symptoms may include dermatitis or inflammatory reactions. If this occurs, discontinue use and obtain medical care immediately. The product should not be used in the presence of, and is not a treatment for, active periodontal disease.

### ► PRECAUTIONS FOR USE

The following information should help dental practitioners reduce the risk of cross-contamination between patients when using multiple-use dental dispensers: apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient; use new, uncontaminated dental gloves when handling multiple-use dental dispensers; utilize dental assistants to dispense material for the dentist; avoid contact of the reusable parts (e.g., the body of the multiple-use dental dispenser) with the patient's mouth; do not reuse the multiple-use dental dispenser if it becomes contaminated; do not reprocess a contaminated multiple-use dental dispenser by using chemical wipes or disinfectants; do not immerse multiple-use dental dispensers in a high level chemical disinfectant, as this may damage the dispenser and the material contained in the device; do not sterilize multiple-use dental dispensers, as this may damage the material contained in the device.

### ► PRODUCT WARNINGS AND CLINICAL INFORMATION

• Store Dryz in a cool dark place prior to use, sealed in the foil package.

• After use, Dryz syringes may be 1) discarded or 2) resealed with the original cap tightly screwed back on, and placed into the foil package, to avoid drying out the material.

• Syringe dispensing needle tips are single-patient use only, and should be discarded after use.

• Unit dose capsules are single-patient use only, and should be discarded after use.

• Avoid sunlight or nearby heat sources. Shelf life will be reduced if not stored properly.

• Dryz Retraction Paste, like many agents used in dentistry to control bleeding, contains Aluminum Chloride. This may cause corrosion on some plastic or metal surfaces (even stainless steel), if left in extended contact. To protect dental instruments, always remember to rinse off Dryz from their surface immediately after each use.

• If product accidentally contacts skin or eyes, wash with copious amounts of water. Obtain medical care immediately if necessary.

• Do not ingest this product, and keep it away from children.

### ► GENERAL INSTRUCTIONS FOR USE

1. After tooth preparation is complete, rinse the sulcus with water. Gently dry with compressed air at an angle to the open sulcus, not directly into it.

2. Open the reusable foil package by tearing off the top where indicated. Save the bag for later storage. Remove the syringe or a unit dose capsule from the package.

3. If dispensing from syringe, remember that Dryz is a thick material being pushed through a thin narrow tip. To ensure that the syringe needle tip does not come off during placement, follow these steps before use.

a. Remove the sealing cap from the syringe. Save the cap, as it may be used to seal the syringe.

b. Depress the plunger to initiate the flow of a small amount of Dryz from the syringe tip. Wipe off the excess.

c. Using the needle tip cover as a wrench, twist a needle tip onto the syringe until it is tight. If the tip is not fully locked on the syringe, the pressure may cause it to come off when the plunger is squeezed.

d. Remove the needle tip cover and carefully bend the needle to the desired angle using any suitable method. If the tip kinks, discard it and use another one.

e. Initiate the flow through the bent needle tip by slowly expressing a small amount of Dryz onto a pad.

4. If using a unit dose capsule, insert it properly into a Centrix-compatible gun, remove the cap and "start" the flow by extruding a small amount onto a pad first. It is recommended that a barrier bag is used with the Centrix-compatible gun.

5. Express Dryz around the tooth into the mouth of the gingival sulcus, until it is slightly overfilled with material. Do not force the tip of either the

needle or the capsule into the tissues. Allow the material to remain undisturbed in the sulcus for 2 minutes.

6. To increase gingival retraction, pre-made cotton compression caps (or standard cotton rolls cut into short sections) may be placed over the tooth, followed by 2 minutes of biting pressure by the patient's opposing teeth.
7. Dryz remains pliable, and does not fully dry out in the sulcus. After 2 minutes, rinse it away with water and air and remove any remaining material using a non-cutting hand instrument or cotton pledge. Rinse and dry again. The sulcus is now ready for the impression or restorative procedure.

8. After use, straighten the needle tip and re-cap it with the needle tip cover. Use the cover as a wrench to remove the tip from the syringe.

9. If any Dryz remains in the syringe, you may reuse it by sealing it with the original plastic cap (not the needle tip), and placing it back into the foil bag. The recapped syringe should be cleaned and disinfected with an EPA-registered (HIV/HBV) claim or intermediate level (tuberkuloid) hospital disinfectant.

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### ► PRODUKTADVARSLER OG KLINISKE OPLYSNINGER

Dryz Gingival Hemostatic Retraction Paste should not be swallowed, and is

contraindicated for use on or by persons who are allergic to or sensitive to aluminum chloride, silicates or similar products. Symptoms may include dermatitis or inflammatory reactions. If this occurs, discontinue use and obtain medical care immediately. The product should not be used in the presence of, and is not a treatment for, active periodontal disease.

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